

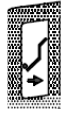
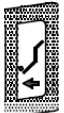
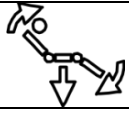


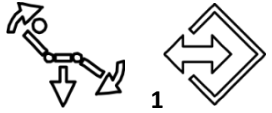

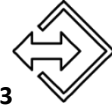

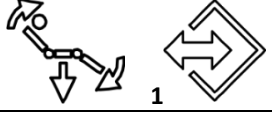





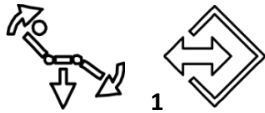

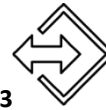
Patient chair control





Backrest buttons	
Manual Command	
Backrest down Push the button towards the right 	Backrest up As long as pressed Push the button towards the left 
Chair down As long as pressed Push the button towards the right 	Chair up As long as pressed Push the button towards the left 

Automatic Command - Standard Version - DENTAL UNITS WITH SOSPENS CUSPIDOR: LINEA ESSE – LINEA PATAVIUM – TEMPO 9 ELX	
Chair to autoposition	
Chair rinsing	

Automatic Command – 3 Memory and Last-Position - DENTAL UNITS WITH FLOOR WATER GROUP: LINEA ESSE – LINEA PATAVIUM – TEMPO 9 ELX	
Chair rinsing/last position	
Chair to autoposition/memory (automatic) The first selection moves to the zero position the chair; pressing the button again it recalls the chair's position 1 memory 	
Memory 2 	Memory 3 

Automatic Command – DENTAL UNITS WITH SOSPENS CUSPIDOR VIRTUOSUS CLASSIC	
Chair rinsing	
Chair to autoposition/memory (automatic) The first selection moves to zero position the chair; pressing the button again it recalls the chair's position 1 memory 	
Select program in instrument console 	Not Available 

Automatic Command – DENTAL UNITS WITH FLOOR WATER GROUP: VIRTUOSUS CLASSIC , CARVING PLUS AND UNIVERSAL ML	
Chair rinsing	
Chair to autoposition/memory (automatic)	The first selection zeroes the chair position, pressing the button again recalls the chair's position 1 memory 
Select program in instrument table	Not Available  

Foot Control			
Backerest down As long as pressed (manual)		Backerest up As long as pressed (manual)	
Chair down As long as pressed (manual)		Chair up As long as pressed (manual)	

Programming

Control panel
Patient chair setting
To memorize a position: move the chair manually up to the desired position (using the manual controls); then press and hold the button of the memory that you want to save until you hear the confirming beep.