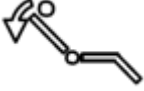

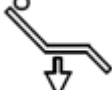






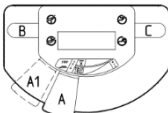




Patient chair control




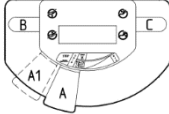

Control panel	
Backerest down As long as pressed (manual)	
Backerest up As long as pressed (manual)	
Chair down As long as pressed (manual)	
Chair up As long as pressed (manual)	
Chair to autoposition/memory (automatic)	The first selection zeroes the chair position, pressing the button again recalls the chair's position 1 memory
Chair rinsing/last position (automatic)	

Foot Control	
Backerest down As long as pressed (manual)	
Backerest up As long as pressed (manual)	
Chair down As long as pressed (manual)	
Chair up As long as pressed (manual)	
Locking Chair Movements	Move the foot control lever A1 to the left and press down.
	

Dental unit control

Control panel	
Cup Fill	(Short activation)
	
Bowl rinse	(Short activation)
	

Instrument control

Control panel		
<p>Spray</p> 		<p>Fiber optic light</p> 
<p>Chip Blower Press level A</p> 		
Micromotor		
<p>Reversing Rotation Direction</p> 		

Programming

Control panel		
Patient chair setting		
<p>Chair to autoposition/memory (automatic)</p>	<p>To store position 1, zero the memory, position the chair in the desired position and keep pressing the button until you hear a beep</p>	