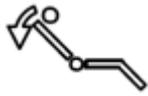
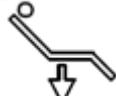
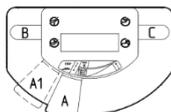


## Patient chair control


Control panel				
<b>Backerest down</b> As long as pressed (manual)			<b>Backerest up</b> As long as pressed (manual)	
<b>Chair down</b> As long as pressed (manual)			<b>Chair up</b> As long as pressed (manual)	

Foot Control				
<b>Backerest down</b> As long as pressed (manual)			<b>Backerest up</b> As long as pressed (manual)	
<b>Chair down</b> As long as pressed (manual)			<b>Chair up</b> As long as pressed (manual)	
<b>Locking Chair Movements</b>			Move the foot control lever A1 to the left and press down.	

## Dental unit control

Control panel		
<b>Cup Fill</b>	(Short activation)	
<b>Bowl rinse</b>	(Short activation)	

# Instrument control

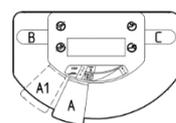Control panel				
Spray			Fiber optic light	
Increase/decrease parameter (power, rpm)		 	Chip Blower	
<b>Turbine</b>				
Speed or fixed / progressive power control				
<b>Micromotor</b>				
Reversing Rotation Direction		Speed or fixed / progressive power control		

# Programming

Control panel	
<u>Cup filling and spittoon rinsing time</u>	
<p>Cup Fill</p>  <p>Press for 3 seconds and release the button when the desired level is reached in the cup.</p>	<p>Spittoon Rinsing</p>  <p>Press for 3 seconds and release the button when the desired rinsing time has elapsed.</p>