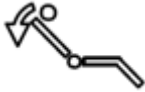

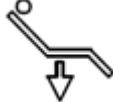

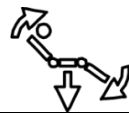

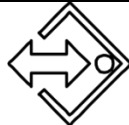





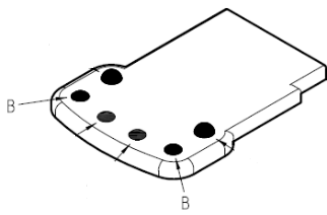
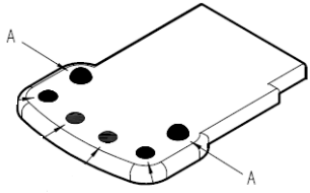
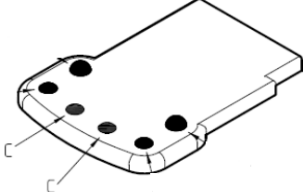


# Patient chair control

Control panel			
<b>Backerest down</b> As long as pressed (manual)		<b>Backerest up</b> As long as pressed (manual)	
<b>Chair down</b> As long as pressed (manual)		<b>Chair up</b> As long as pressed (manual)	
<b>Chair to autoposition</b> (automatic)		<b>Chair rinsing/last position</b> (automatic)	
<b>Memory 1</b> (automatic)		<b>Memory 2</b> (automatic)	
<b>Locking Chair Movements</b>		<b>OK</b>	Press the key for approx. 3 seconds until a beep is heard to disable the dental chair
<b>Chair Swivelling</b> (ONLY FOR TOP VERSION)		<b>+</b>	To unlock the dental chair swiveling movement, press the key and turn the chair in the desired direction (clockwise or anticlockwise).

Foot Control			
<b>Backerest down</b> As long as pressed (manual)		<b>Backerest up</b> As long as pressed (manual)	
<b>Chair down</b> As long as pressed (manual)		<b>Chair up</b> As long as pressed (manual)	

Chair Platform (ONLY FOR CARVING VERSION)	
<b>Recall the memory that was previously activated</b> (automatic)	<b>Chair to autoposition</b> (automatic)
	
<b>Chair rinsing/last position</b> (automatic)	
	

# Dental unit control

Control panel		
Operating lamp		
Cup Fill	(Short activation)	
Bowl rinse	(Short activation)	
Door open / assistant call function		
Time /date settings		









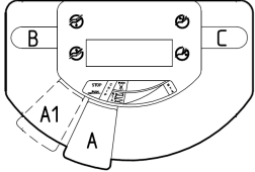
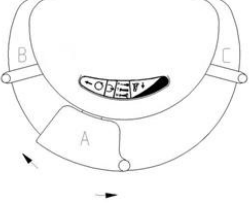
## Foot Control

Function can be programmed

# Instrument control

Control panel		
User profile	→	
	press until "CURRENT USER No." appears	Adjust value
Spray Clean Spray		Chip Blower Press level A 
		Fiber optic light 
Increase/decrease parameter (power, rpm, transmission ratio, torque, etc.)		Confirm parameter 
Turbine		
Speed or fixed / progressive power control		
Scaler (Enabling the SCALER, PERIO AND ENDO functions depends on the installed scaler model.)		
Scaler function	Perio function	Endo function

# Instrument control

<b>Control panel</b>			
<b>Micromotor</b>			
Memory 1		Memory 2	
Memory 3			
Speed or fixed / progressive power control			
<b>Micromotor Brushless</b>			
Memory 1		Memory 2	
Memory 3			
Speed or fixed / progressive power control		(*)Fast mode	<b>FAST</b>
(*)Auto Forward mode	<b>AUTO FORWARD</b>	(*)Transmission Ratio	<b>RATIO 1X</b>
(*)Auto Reverse mode		(*)Maximum Torque	<b>TORQUE</b>
<b>Reversing Rotation Direction</b> Move the foot control lever A to the left and press it downwards until a beep is heard.			
<b>Reversing Rotation Direction</b> move the central lever A to the left until a beep is heard.			

(\*) Only for enabled brushless micromotor

# Programming

<b>Control panel</b>	
<b>User profile setting</b>	
<p>User profile</p> <p>press until "CURRENT USER No." appears      Adjust value      Accept new value</p>	
<b>Patient chair settings</b>	
<p>Memory 1-2-3 (automatic)</p>	<p>To save a dental chair position, move the chair to the required position and hold down the key until you hear the beep that indicates that the position has been saved.</p>
<p>Chair to autoposition (automatic)</p>	<p>To save a dental chair position, move the chair to the required position and hold down the key until you hear the beep that indicates that the position has been saved. This enables storage of the pantographic arm reset position.</p>
<p>Chair rinsing (automatic)</p>	<p>To save a dental chair position, move the chair to the required position and hold down the key until you hear the beep that indicates that the position has been saved. This enables storage of the rinse position.</p>
<b>Instrument settings</b>	
<p>Instrument Lighting</p> <p>Adjust value      Accept new value</p>	<p>Spray</p>
<b>Cup filling and spittoon rinsing time</b>	
<p>Cup Fill</p> <p>Press for 3 seconds and release the button when the desired level is reached in the cup.</p>	<p>Spittoon Rising</p> <p>Press for 3 seconds and release the button when the desired rinsing time has elapsed.</p>
<b>Micromotor settings</b>	
<p>Memory 1-2-3</p> <p>Select programs      Adjust value      Press for few seconds (beep)</p>	
<b>Clock and data settings</b>	
<p>Clock and data</p> <p>Adjust value      Confirm new value</p>	